

- ✓ Fast results only in 10 minutes
- ✓ No special equipment or training required
- ✓ All-in-one package

Instructions For Use

Please comply with the latest Therapeutic Goods Administration (TGA) conditions for the use of the Rapid Antigen Test, as advised by Pantonic Health.

Test Contents

- 1 TEST DEVICE
- 1 STERILE ANTERIOR NASAL SWAB
- 1 EXTRACTION VIAL
- 1 EXTRACTION VIAL CAP


You will also need

- HAND SANITISER
- A FACE MASK
- MEDICAL GLOVES
- A BIOHAZARD WASTE BIN



01.


Remove the test device from its packaging, placing it on a clean, flat surface.

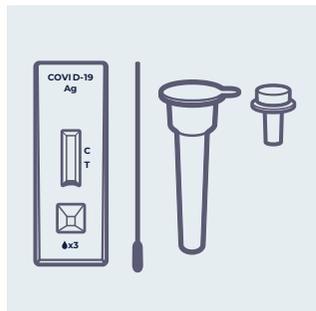

02.

Locate the nasal swab and carefully remove it from its packaging, ensuring not to touch the swab tip.


Gently insert the swab 2 - 3cm into your nostril.

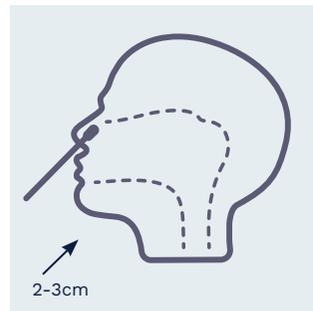

03.

Slowly rotate the swab at least 5 times in a circular path over the surface of the nostril for 15 seconds.

Using the same swab, repeat on your other nostril.

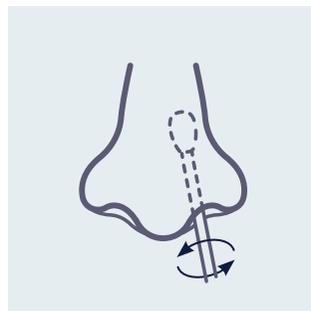


04.

Locate the extraction vial and gently peel off the aluminium foil seal.




05.

Carefully place the swab into the extraction vial and rotate the swab vigorously at least 5 times.

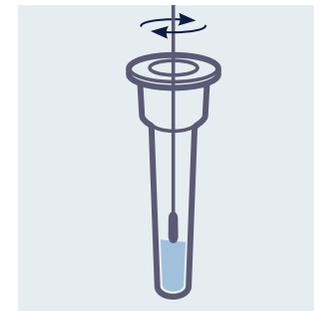

06.

Remove the swab by rotating against the extraction vial, while squeezing the sides of the vial to release the liquid from the swab.

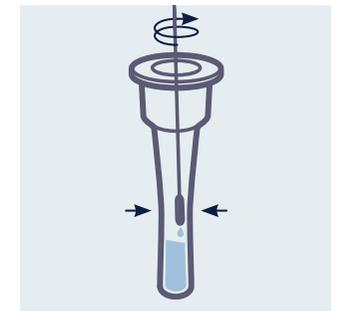


- ✓ Fast results only in 10 minutes
- ✓ No special equipment or training required
- ✓ All-in-one package

Instructions For Use (continued)


07.

Close the vial by firmly pushing in the cap provided.

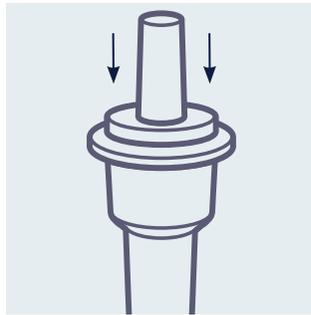

08.


Thoroughly mix the vial by swirling and flicking the bottom of the tube.

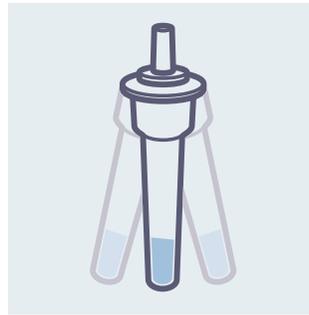

09.

Invert the extraction vial above the sample well on the test device.

Squeeze the vial gently and allow 3 drops of sample to fall into the sample well.

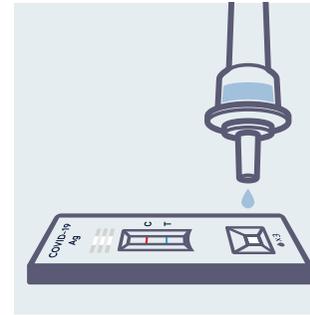

10.

Wait for 10 minutes for the results. Do not move or lift during this time.

Results should not be read before 10 minutes or after 15 minutes.



11.

Dispose all testing components, including the test device, in an appropriate hazardous waste bin.




Reading The Results

**COVID-19
positive**


- A red line at 'C'
- A blue line at 'T'

Any faint coloured line should be considered as positive.

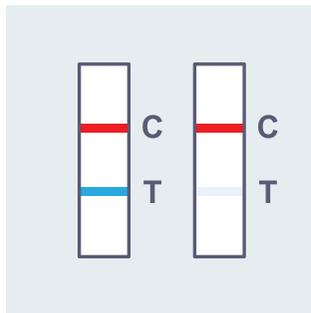


**COVID-19
negative**

- A red line at 'C' only

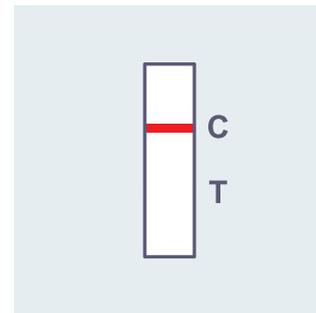

**Inconclusive
result**

If the red line is not visible, the result is invalid. Another test will be required.

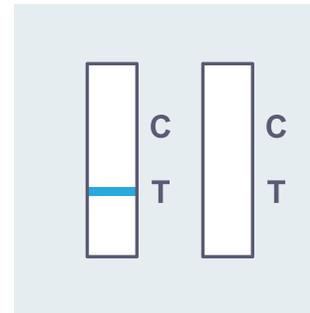

**In the case of
a positive result,
you must
immediately**

01.

Undergo a laboratory-based PCR test

02.

Contact your local Public Health Unit and follow their instructions

03.

Self-isolate until you receive a negative PCR test result or as instructed